

**Sidekick™ Sharpener** **A202**

**Diamond Sharpening Cards** **A204**

**Stones** **A205**

Arkansas A205


India A206

Ceramic A206

**Composition Stones** **A207**

**Accessories** **A208**

Grips A209

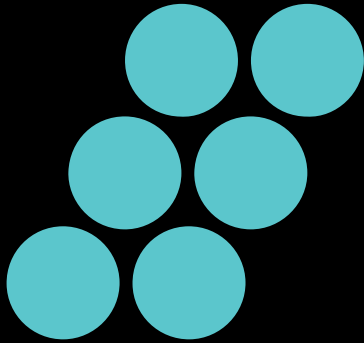

## **Instrument Maintenance**

Regular maintenance is vital to longevity and performance of Instruments. Scalers and cures should be kept sharp and true to their original design. Dental procedures are most effective when using sharp instruments because they reduce fatigue, improve deposit removal, save time, enhance tactile sensitivity, and minimize patient discomfort.



# Sidekick™ Sharpener

The Sidekick™ Sharpener is designed for routine maintenance sharpening of scalers and cures. Instrument guide channels help control blade angulation, providing consistent sharpening results. Requiring 2 AA batteries, Sidekick is compact and cordless, allowing for use anywhere in the dental office.



Learn more about our [Sidekick™ Sharpener](#).



#### SideKick™ Sharpener Kit

[SDKKIT](#)

#### Sidekick™ Replacement Stones

Ceramic Coarse Grit [SDKSTCOARS](#)

Arkansas [SDKSTONEA](#)

Ceramic [SDKSTONEC](#)

#### Sidekick™ Replacement Guide Plate

[SDKGPLATE](#)

#### Sidekick™ Replacement Screw

[SDKGPSCREW](#)

#### Sidekick™ Replacement Stoneclip

[SDKCLIP](#)

**Cross  
Sharpening  
Off the  
*To-Do* List**

**That's a  
Great Idea**



**EverCare**

SCALER SHARPENING SERVICE



**EASY ORDER  
MANAGEMENT**



**SAVE TIME  
AND MONEY**



**FAST  
TURNAROUND**



Enroll today at [EverCareSharpening.com](https://EverCareSharpening.com)

**HuFriedyGroup**  
The Best In Practice



# Diamond Sharpening Cards

Diamond Sharpening Cards feature a uniform coating of industrial diamonds bonded to a stainless steel card providing a hard, flat sharpening surface that resists ditching and grooving. They also feature easy clean-up via ultrasonic cleaner and steam sterilization.



Learn more about our [Diamond Sharpening Cards](#).



**Diamond Sharpening Cards Set**  
*Includes 1 each: Extra Fine, Fine and Medium Grip Diamond Sharpening Card.*

**DSCSET**

**Diamond Sharpening Cards**

Extra Fine Grit	<b>DSCXFINE</b>
Fine Grit	<b>DSCFINE</b>
Medium Grit	<b>DSCMED</b>

# Stones

From Extra Fine to Coarse grit levels, HuFriedyGroup offers a complete selection of natural and synthetic sharpening stones to help you maintain the perfect edges on your dental instruments.

## Arkansas

Natural Arkansas stones are available in fine and super fine grits - ideal for daily sharpening of dental instruments. Arkansas stones should be coated with a thin layer of sharpening oil prior to use in order to prevent clogging and ensure top performance.




<b>Arkansas Stone Sharpening Kit</b>	<b>SSKITA</b>
#4 Flat Arkansas Stone, Fine Grit	<b>SS4</b>
#299 Arkansas Conical Stone, Super Fine Grit	<b>SS299</b>
Plastic Test Stick, 6 Pack	<b>PTS</b>
Sharpen-Ez Stone Oil, 1 fl.oz.	<b>SSO</b>
Magnifying Lens Credit Card Size	<b>LENS</b>
HVESOLO™ Disposable Evacuation Tips (2.5" length)	<b>1808S</b>



#AS-22 Arkansas Stone, Super Fine Grit  
**SSAS22**



#4 Flat Arkansas Stone, Fine Grit  
**SS4**

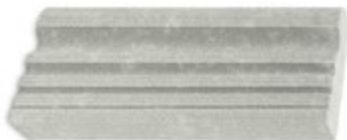

#5 Bates Arkansas Stone, Fine Grit  
**SS5**



#6A Arkansas Stone (Wedge), Fine Grit  
**SS6A**



#2 Arkansas Cylindrical Stone, Super Fine Grit  
**SS2**



#299 Arkansas Conical Stone, Super Fine Grit  
**SS299**



### India

India or I-Stones are synthetic stones available in fine and medium grits. I-stones can be lubricated with water prior to use or can be used dry.



#1 I-Stone, Fine Grit  
**SSF1**



#6 I-Stone (Wedge) Medium Grit  
**SS6**



#22 I-Stone, Medium Grit  
**SSMS22**



#309 I-Stone, Tapered, Fine Grit  
**SS309**

### Ceramic

Hard synthetic stones available in fine, medium and coarse grits. Excellent for routine sharpening of dental instruments. Use water for lubrication or may be used dry.



<b>Ceramic Stone Sharpening Kit</b>	<b>SSKITC</b>
#3 Ceramic Stone, Medium Grit	<b>SS3C</b>
Ceramic Cylindrical Stone, Fine Grit	<b>SSKC</b>
Plastic Test Stick, 6 Pack	<b>PTS</b>
Magnifying Lens Credit Card Size	<b>LENS</b>



**Dual Grit Ceramic Stone**  
*Features a coarse side for recontouring and a finer side for finishing and honing.*  
**SSDGRIT**



#3 Ceramic Stone, Medium Grit  
**SS3C**



Ceramic Cylindrical Stone, Fine Grit  
**SSKC**



Ceramic Cylindrical Stone for Luxating Elevators, Fine Grit  
**SSLUX**

# Composition Stones

Composition stones are coarse aluminum oxide mandrel mounted stones for the handpiece, designed to restore the cutting edge on dull instruments.



Learn more about our [Composition Sharpening Stones](#).



**#300 HP Composition Stone**  
(6 x 6mm) at base  
**SS300**



**#301 HP Composition Stone**  
(6 x 10mm) diameter  
**SS301**



**#302 HP Composition Stone**  
(3 x 3mm) diameter  
**SS302**



**#303 HP Composition Stone**  
(6 x 13mm) diameter  
**SS303**



**#304 HP Composition Stone**  
(6 x 16mm) diameter  
**SS304**



**#305 HP Composition Stone**  
(6 x 19mm) diameter  
**SS305**





# Accessories



**Carbon Steel File Sharpener**  
**FS**



**Carbon Steel Tanged File Sharpener**  
**FST**



**Magnifying Lens Credit Card Size**  
**LENS**



**Sharpen-Ez Stone Oil, 1 fl.oz.**  
**SSO**



**Stainless Steel Pumice Cup**  
**PCUPM**



**Plastic Test Stick**  
QTY: 1 **PTS-IND**  
QTY: 6 **PTS**

**Measuring Cup with 25ml Markings**  
**MCUPE**

## Grips

Grips are used to identify and organize instruments.



### Signature Series Grips (80)

*Used with the #7, #8, #C8 and #9 handles*

Assorted **SSGMULTI**

### Signature Series Grips (16)

*Used with the #7, #8, #C8 and #9 handles*

● Gray	<b>SSG1</b>
● Pink	<b>SSG10</b>
● Lavender	<b>SSG11</b>
● Ocean Blue	<b>SSG12</b>
○ White	<b>SSG2</b>
● Orange	<b>SSG3</b>
● Brown	<b>SSG4</b>
● Yellow	<b>SSG5</b>
● Red	<b>SSG6</b>
● Green	<b>SSG7</b>
● Blue	<b>SSG8</b>
● Black	<b>SSG9</b>
Assorted	<b>SSG0</b>

### ResinEight™ Colors Grips (16)

*Used with the #7, #8, #C8 and #9 handles*

● Blue	<b>SSGBLUE8</b>
● Gray	<b>SSGGRAY8</b>
● Green	<b>SSGGREEN8</b>
● Lavender	<b>SSGLAVNDR8</b>
● Orange	<b>SSGORANGE8</b>
● Purple	<b>SSGPURPLE8</b>
● Red	<b>SSGRED8</b>
● Yellow	<b>SSGYELLOW8</b>